# *STERILUX XL*

manuel d'instructions  
**instruction** manual

SILHOUET-TONE<sup>®</sup>

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**WARNINGS :** To reduce the risk of electric shock, do not expose this product to water or moisture.

**READ ALL INSTRUCTIONS BEFORE USING APPLIANCE.**

**WARNING: ULTRAVIOLET RADIATIONS – FOLLOW INSTRUCTIONS.**

## 01. Operating instructions

- Open cover and carefully set the accessories on the tray, inside the unit.
- Gently close the cover and switch the unit ON.
- Sanitize accessories for 20 minutes. Avoid contact between, or overlapping of the accessories to be sanitized.
- Turn unit OFF and remove accessories from Sterilux. Unplug the unit after use.
- Since most accessories are sterilized when wet, it's a good idea to wipe the bottom dry with a cloth after each use. The chance of mold growing on the bottom is slim since it is killed by the heat or ultraviolet lamp used in the sterilizer.

## 02. Description

- The Sterilux XL is specifically designed for the disinfection of glass electrodes, brushes, scissors and/or any other accessories useful to the professional aesthetician. Spacious and elegant, the Sterilux XL allows the disinfection of more than one accessory, at the same time.
- The unit is highly recommended for hygienic purposes to avoid any problems or diseases transmitted by carelessly cleansed accessories.
- Sterilux XL is equipped with a germicidal, ultraviolet lamp (#411718). An ultraviolet lamp has a life span of about 6 000 hours depending on the manufacturer, size and hours of usage. Always keep a spare available with your supplies.
- A safety fuse is located at the back of the unit. If the fuse needs to be replaced, use the part number provided in the technical specifications.
- **IMPORTANT NOTICE.** For maintenance of your acrylic accessories or equipment, it is recommended to clean it with soapy water. The frequent use of glass cleaner (ex.: Windex) produces permanent scratches on the acrylic surface.

## 03. Technical specifications

|                   |   |              |
|-------------------|---|--------------|
| Power supply      | 120 or 220 Volts                            |              |
| Power consumption | 15 Watts                                    |              |
| Cycles            | 50 – 60 Hz                                  |              |
| Fuse              | 1 A FB #23912006                            |              |
| Dimensions        | Length :                                    | 15" (385 mm) |
|                   | Width :                                     | 10" (265 mm) |
|                   | Height :                                    | 8" (210 mm)  |
| Weight            | 9 lbs (4 kg)                                |              |
| Materials         | Metal, tinted Plexiglas, powder coat paint. |              |

Silhouet-Tone reserves the right to change the specifications of the apparatus without notice.

All rights reserved. No part of this work covered by the copyright hereon may be reproduced or used in any form or by any means – graphic, electronic, or mechanical, including photocopying, recording, taping, or information storage and retrieval systems – without the permission of the company Silhouet-Tone Ltd.

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**SILHOUET-TONE (CANADA) LTD**
**SILHOUET-TONE (U.S.A.) LTD**
St. Albans, Vermont 05478 U.S.A.
**SILHOUET-TONE (EUROPE) S.A.**